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## Instructions for Use EN

### Dual cure composite with nano-zirconium dioxide for core build-up and post cementing

#### Product description

**Harvard ZirconCore** is a dual cure high radiopaque flowable microhybrid composite material with nano particles (nano-zirconium dioxide, nanocalciumfluoride) for core-build-up and cementation of posts. Due to its excellent mechanical properties final crown preparation can be carried out more precise. Harvard ZirconCore is also suitable for cementing of crowns and bridges, inlays and onlays. For all indications the use of a dual cure bonding agent (e.g. Harvard Bond SE Dual) is required before application of the composite.

Harvard ZirconCore is based on poly- and difunctional methacrylates and inorganic filler particles of 0.02-10 µm. The total filler content is 64 % by weight and 48 % by volume.

Delivered in 1:1 Minimix-syringes Harvard ZirconCore can be easily dispensed and applied directly. Harvard ZirconCore exhibits a short setting time without high heat generation. The dual cure properties enables the dentist, also to carry out cementations and core build-ups in cases where a light cure cannot be guaranteed to be sufficient.

#### Indications/intended use

- Cementing of posts
- Core build-ups
- Cementing of crowns and bridges, inlays and onlays

#### Performance features

The performance features of the product meet the requirements of the intended use.

#### Contraindications

The placement of Harvard ZirconCore is contraindicated if a dry working area or the recommended application technique are not possible. Irritations resulting from direct contact with the pulp cannot be ruled out. Therefore for pulp protection areas close to the pulp should be covered with a thin layer of a pulp capping material.

#### Patient target group

Persons who are treated during a dental procedure.

#### Intended users

This medical device should only be used by a professionally trained dental practitioner.

#### Incompatibility with other materials

Do not use in combination with substances containing eugenol because eugenol inhibits the polymerization of the composite. Neither store the composite material in proximity of eugenol containing products, nor let the composite allow coming into contact with materials containing eugenol.

#### Application

##### 1. Preparing the Minimix-Syringe

Harvard Dental International GmbH only recommends for Harvard ZirconCore the use of mixing tips type Harvard Mini 1:1 O-Brown.

Remove the cap of the Minimix-syringe and throw it away (**do not use it again!**). It is replaced by a supplied 1:1 mixing cannula with an intra tip long. Turn the cannula 90° until it locks in position. The Material is now ready for application.

#### Note:

Discard the first 2-3 mm of the extruded material (about the size of a peppercorn). This must be done for each new mix.

Store used syringe with fixed used mixing cannula in the dark.

**The working time (23°C (74°F)) of Harvard ZirconCore in the self cure mode is 1:30 minutes from start of mixing.**

#### 2. Post cementing

##### 2.1. Isolation

Use of a rubber dam to isolate the tooth is strongly recommended.

##### 2.2. Root canal preparation

Refer to directions of the selected post manufacturer.

Remove all existing old restorations and decay from the tooth. Prepare and clean the root canal with e.g. sodium hypochlorite solution, rinse and remove excess solution from the canal with a soft paper tip.

Conditioning of the root canal should be done with the self etching bonding agent Harvard Bond SE Dual:

One drop of Harvard Bond SE Dual PART A and one drop of Harvard Bond SE Dual PART B were combined in a mixing pallet and mixed for **5-10 seconds**.

#### Note:

Do not interchange lids of the bottles, because this can lead to a cross-contamination of the liquids.

Apply the homogeneous mixture generously with a brush onto the slightly wet root canal walls for **15 seconds** with agitation. Repeat procedure 1 –

2 times. All the dentine surfaces must kept wet with the primer over the specified time

Remove excess Harvard Bond SE Dual with dry paper points. Dry the root canal cautiously with oil free air for **15 seconds** to remove all volatile components and to disperse the adhesive to an even layer. Light cure all areas that are available for a dental curing unit for **20 seconds. For a maximal adhesion it is strongly recommended to use the light cure mode.**

If light cure is absolutely impossible, the adhesive will also cure in the autocure mode. After application of the adhesive air thin to **remove all volatile components**. There must remain a sticky layer.

#### 2.3. Cementing of the post

Prepare the selected post according to manufacturer directions.

Harvard ZirconCore is applied into the prepared root canal and onto the post. Seat the post careful into the canal and maintain firm pressure until the post is seated. Harvard ZirconCore self-cures within **3:30 minutes**. For post stabilization light cure the coronal part of the cemented post for **20 seconds** with a polymerization unit (wavelength range 400-500 nm) with a light intensity of at least 1000mW/cm<sup>2</sup>.

As soon as the Harvard ZirconCore has set proceed with the core-build-up procedure.

#### 3. Core-Build-Up

##### 3.1 Isolation

Use of a rubber dam to isolate the tooth is strongly recommended.

##### 3.2. Cavity Preparation

Remove all existing old restorations and decay from the tooth. If necessary place any pins or posts. Refer to directions of the selected post manufacturer.

##### 3.3. Pulp Protection

For pulp protection areas close to the pulp should be covered with a thin layer of pulp capping material (e.g. Harvard BioCal®-CAP or Harvard CalciumHydroxide).

##### 3.4. Application of a Bonding Agent

Conditioning should be done with the self etching bonding agent Harvard Bond SE Dual:

One drop of Harvard Bond SE Dual PART A and one drop of Harvard Bond SE Dual PART B were combined in a mixing pallet and mixed for **5-10 seconds**.

#### Note:

Do not interchange lids of the bottles, because this can lead to a cross-contamination of the liquids.

Apply the homogeneous mixture generously with a brush onto the slightly wet enamel- and dentin surfaces for **30 seconds** with agitation. The material should build a homogeneous layer. Air thin for **10 seconds** to remove the volatile components and to disperse the adhesive. Then light cure with a dental halogen light unit for **20 seconds** and place Harvard ZirconCore. **For a maximal adhesion it is strongly recommended to use the light cure mode.**

If light cure is absolutely impossible, the adhesive will also cure in the autocure mode. After application of the adhesive air thin to **remove all volatile components**. There must remain a sticky layer.

#### Note:

It is essential that the primed dentine and enamel surfaces are dry and contaminant free for the application of Harvard ZirconCore.

#### 3.5. Application

Place the mixing cannula directly into the preparation and press out the paste.

Harvard ZirconCore is automatically mixed when dispensed with slight and even pressure. Filling should occur from bottom upwards to prevent voids. To facilitate placement of Harvard ZirconCore place a matrix band around the prepared tooth.

Harvard ZirconCore may be contoured by using a composite instrument. A flat-ended interproximal carver is recommended.

Place Harvard ZirconCore directly into the preparation and allow the system to self cure for **3:30 minutes**. After that the material should be light-cured (**40 seconds**) with a polymerization unit (wavelength range 400-500 nm) with a light intensity of at least 1000mW/cm<sup>2</sup>. With this technique an optimum of physical properties will be obtained.

An explorer can be used to test that the Harvard ZirconCore has completely set. Remove the matrix not earlier than the material has set.

Final preparation on the Harvard ZirconCore can be carried out by using crown preparation burs.

#### 4. Additional informations/Warnings

- The ambient light of the dental lamp may start polymerization of the composite.
- Do not use any resin to adjust viscosity of composite restorative material.
- Avoid contact with skin, mucous membrane and eyes.
- If the material comes into contact with skin, immediately wash with water and soap. If the material comes into contact with eyes, immediately rinse with copious amounts of water and seek medical advice if required.
- Unpolymerized composite may have an irritant effect and can lead to sensitization against methacrylates.
- Color stability meets the requirements of DIN EN ISO 4049.
- Commercial medical gloves do not protect against the sensitizing effect of methacrylates.
- Keep away from children!

#### Storage information

Do not store above 20 °C (68 °F). Protect from direct sunlight. Store unopened material in the refrigerator. Opened cartridges have to be used up within 3 months. Do not use after expiry date.

#### Composition

Dimethacrylates, dental glass, silicon dioxide, photoinitiators, catalysts.

#### Disposal

Disposal of the product according to local authority regulations.

#### Reporting obligation

Serious incidents according to the EU Medical Devices Regulation that have occurred in connection with the medical device must be reported to the manufacturer and the competent authority.

#### Note

The summary of safety and clinical performance of the medical device can be found in the European database on medical devices

(EUDAMED – <https://ec.europa.eu/tools/eudamed>).

#### Warranty

Harvard Dental International GmbH warrants this product will be free from defects in material and manufacture. Harvard Dental International GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and Harvard Dental International GmbH's sole obligation shall be repair or replacement of the Harvard Dental International GmbH product.

#### Limitation of liability

Except where prohibited by law, Harvard Dental International GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

#### Harvard ZirconCore conforms to the DIN EN ISO 4049, type 2, class 3.

Keep the instructions for use for the duration of the application.

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