




HARVARD®

Harvard LuteCem SE

Self-adhesive, dual cure resin-modified luting cement

Harvard LuteCem SE enables a fast workflow – no etching and bonding required. Comfortable application into the root canal through IntraTips. Easy removal of excess material. In addition, Harvard LuteCem SE has a high fluoride release and excellent color stability.

- >> Suitable for:
 1. Luting of posts made of fiber-reinforced materials, metal and ceramics
 2. Luting of crowns and bridges made of ceramics, zirconia, composites and metal
- >> Dual cure
- >> Fluoride release
- >> High color stability

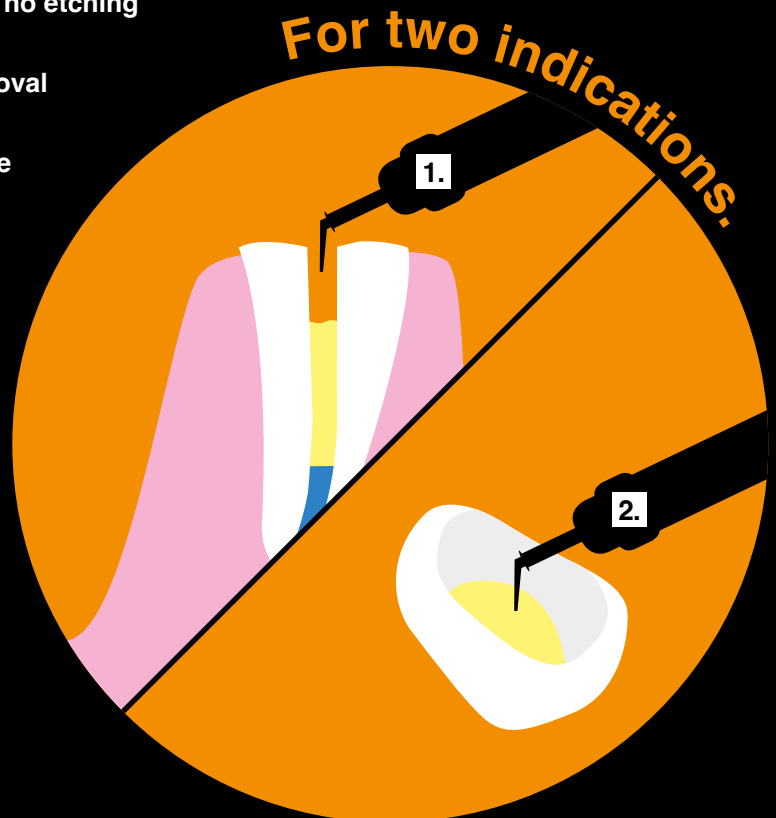


Easy to use
and time
saving!



Always SECURELY cemented.

For two indications:

1. Post cementation

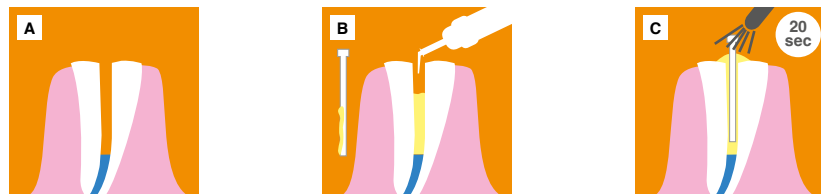


A: Prepared root canal for post cementation
B: Application of **Harvard LuteCem SE** onto the post and into the prepared root canal
C: Positioning of the post into the root canal, light curing of the coronal part

2. Crown cementation




A: Application into the crown
B: Crown seated and light cure at the cervical edge
C: Result: Easy removal of material excess

Physical Properties

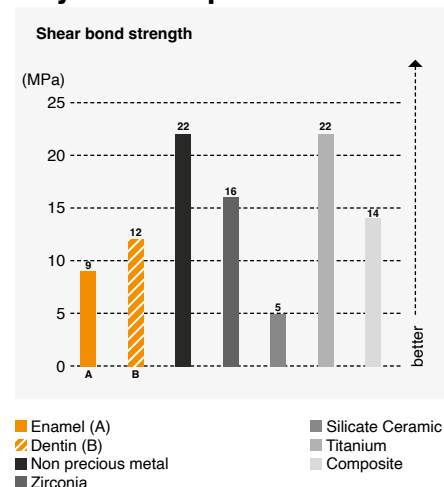

Technical data	
Shear bond strength on enamel/dentin	9/12 MPa
Compressive strength	180 MPa
Setting time (self cure)	4:00 min.
Light cure (optional)	20 sec

Cumulative fluoride release

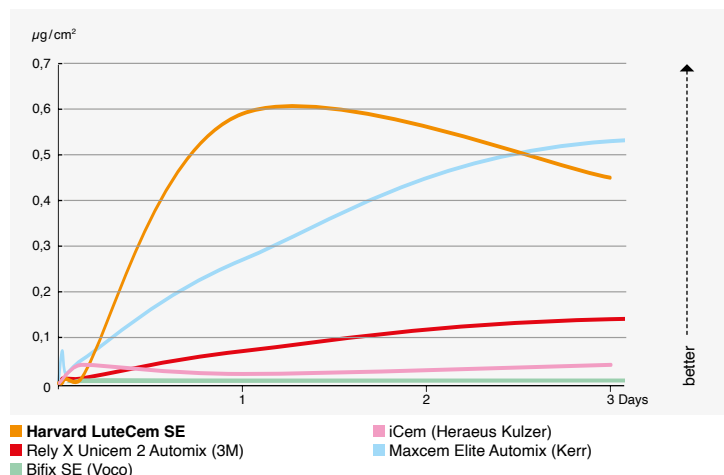

*Much more
than you
expect.*

Order Details

5 ml minimix syringe, 10 mixing tips Harvard Mini 4:1 / 10:1 S-Brown 5 mixing tips Harvard Mini 4:1 / 10:1 O-Brown 5 intra tips long	
Translucent	7081101
A2	7081105
White opaque	7081107
Harvard Mini 4:1 / 10:1 S-Brown Refill bag with 50 mixing tips	7093050
Harvard Mini 4:1 / 10:1 O-Brown Refill bag with 50 mixing tips	7091100
Harvard IntraTips long Refill bag with 50 intra tips	7083620

No stock item, please contact the Harvard sales team.

Harvard Distribution Partner.



HARVARD®

Marke und Qualität seit 1892

Harvard Dental International GmbH
Margaretenstr. 2 - 4, 15366 Hoppegarten, Germany
Phone: + 49 (0) 30/99 28 978-0
Fax: + 49 (0) 30/99 28 978-19
info@harvard-dental-international.de



www.harvard-dental.de