## Harvard Cem OptiCaps®

Zinkphosphat-Zement 

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## Harvard Cem OptiCaps®

Zinc phosphate cement 

### Instruction for use for the dentist

#### Composition

Powder contains: zinc oxide, magnesium oxide, pigments;  
Liquid contains: phosphoric acid. 

#### Areas of applications

Harvard Cem OptiCaps® zinc phosphate cement is qualified for cementing of crowns, bridges, inlays, onlays and orthodontic bands.

Capsules are easily activated and their content is easily ejected out with an extrusion gun.  
Capsule mixing (mixing time 10 seconds) is achieved by a high frequency mixer with about 4,300 oscillations/min.  
Application can be done directly from the capsule.

#### Activation and mixing of Harvard Cem OptiCaps® (see instructions on the back side)

Activate and mix the capsule according to the instructions.

#### Application

Apply the mixed cement to the prepared bonding surface of the restoration.  
Seat immediately (**working time approx. 3 minutes** from start of mixing at 23 °C (73 °F)).

**Net setting time** of Harvard Cem OptiCaps® zinc phosphate cement is **maximum 8 minutes**.

#### Special information

- The mixed cement should never be applied directly onto dentine near to pulp or onto exposed pulp.
- Irritations of the pulp may occur following the cementing.
- The cement must be applied to the dry tooth surface and must set without contact to moisture.
- The liquid contains phosphoric acid and is corrosive. In case of accidental eye contact immediately rinse with large quantities of water.

#### For professional use only!

**Useable volume of mixed cement per capsule**  
0,13 to 0,14 ml.

#### Package sizes

Boxes of 10 capsules and 50 capsules available.

#### Harvard Cem OptiCaps® zinc phosphate cement

- Do not use after the expiration date.
- Storage temperature: 2 – 28 °C.
- Compliant with DIN EN ISO 9917.

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