Harvard Applier OptiCaps®

Harvard Applier OptiCaps®

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- Harvard Cement OptiCaps®
- Harvard MTA OptiCaps® (all types)
- Harvard Ionoglas Cem Extra OptiCaps®
- Harvard Ionoglas Fill Extra OptiCaps®
- Harvard Ionoresin Cem Extra OptiCaps®
- Harvard Ionoresin Fill Extra OptiCaps®

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The capsule applier is for activation and dispensing of materials in Harvard OptiCaps®.

Sterilization instructions

Cleaning and disinfecting

Remaining cement should be removed immediately.

Clean the capsule applier with a gauze soaked in alcohol or in an ultrasonic bath. Chemical disinfectant and sterilization agents can be used according to manufacturer instructions.

Note: It is recommended to use a disinfectant solution which contains a rust inhibitor additive.

Sterilization

The capsule applier can be autoclaved at 134 °C - 137 °C (273 °F - 279 °F) for 3 - 3.5 minutes. Please follow the instructions of the manufacturer.

Note

Sterilize the capsule applier prior to use.
Keep the capsule applier in a dry place to prevent from corrosion.

Warranty

Harvard Dental International GmbH warrants this product will be free from defects in material and manufacture. Harvard Dental International GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and Harvard Dental International GmbH's sole obligation shall be repair or replacement of the Harvard Dental International GmbH product.

Limitation of liability

Except where prohibited by law, Harvard Dental International GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

For dental use only!

Store product out of reach of children!

Keep the instructions for use for the duration of the application.

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