Harvard PremiumSil Putty Soft

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Instructions for Use EN
Vinylpolysiloxane impression material, Putty, kneadable

Harvard PremiumSil Putty Soft is used as a base impression material for the (one step) two phase impression technique in conjunction with Harvard PremiumSil Medium or for the putty/wash technique in conjunction with Harvard PremiumSil Light.

Harvard PremiumSil Putty Soft is a soft but not sticky, kneadable precision impression material based on addition curing vinyl silicones. Harvard PremiumSil Putty Soft is easy to mix and shows good setting properties.

Harvard PremiumSil Putty Soft meets the requirements of ISO 4823, type 0 (putty consistency).

Contraindications/side effects
 None known

Application

1. Mixing

Perforated or non-perforated metal or plastic trays may be used with Harvard PremiumSil Putty Soft. In order to improve adhesion of Harvard PremiumSil Putty Soft to the tray, a silicone adhesive (e.g. Harvard TraySive) should be applied according to the corresponding instructions.

Use equal parts (volume or weight) of base paste and catalyst paste. Take the base paste with the dark measuring spoon and the catalyst paste with the white measuring spoon. Knead until the color is homogeneous (at least **30 seconds**).

2. Impression taking

The ready kneaded material is to load into the prepared tray. Parallel mixed material of Harvard PremiumSil Medium is filled into a syringe or applied directly from the automix-cartridge at approximal and gingival areas and on top of the Harvard PremiumSil Putty Soft on the tray. Insert the loaded tray in the mouth not later than **1:15 minutes** (working time at 23°C or 74°F) after start of mix. Maintain position for at least **2:30 minutes**.

3. Casting

The impression should be stored under normal room conditions (max. 25°C or 77°F). The impression can be poured 30 minutes after removal from the mouth. No other time limitation does exist.

4. Electroplating

All impressions can be copper- or silverplated.

5. Disinfection

Impressions can be disinfected with standard disinfection solutions.

Notes:
 Do not contact the product with latex gloves, they may inhibit the setting. Do not touch teeth with latex gloves.

Do not use with standard (condensation reaction type) silicone impression materials.

Storage

Do not store above 25 °C (77 °F). Do not use after expiry date.

Warranty

Harvard Dental International GmbH warrants this product will be free from defects in material and manufacture. Harvard Dental International GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and Harvard Dental International GmbH's sole obligation shall be repair or replacement of the Harvard Dental International GmbH product.

Limitation of liability

Except where prohibited by law, Harvard Dental International GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

For dental use only!

Store product out of reach of children!
 Keep the instructions for use for the duration of the application.

Order-No.	Article
7083741	Harvard PremiumSil Putty Soft, 2 jars ea. 600g, 2 spoons

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