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Bestell Nr. Artikel

Harvard TEMP Cem EasyDose®
7081103 10 ml EasyDose®-Spritze

Eugenol-free temporary cement for cementing crowns and bridges

Harvard TEMP Cem EasyDose® is a non-eugenol temporary crown and bridge cement suitable for cementing temporary crowns or bridges as well as for trial cementing of permanent restorations. Harvard TEMP Cem EasyDose® shows excellent adhesion to temporary crown and bridge materials, but can be easily removed from prepared teeth.

Harvard TEMP Cem in EasyDose®-syringes is very easy and comfortable to use.

Due to its non-eugenol formulation it does not influence the setting of resin-based filling and luting materials. The temporary crown and bridge cement is based on zinc oxide and organic acids. Especially in cases of allergies to eugenol it is strongly recommended to use this eugenol-free type of cement.

Contraindications

If the patient is known to be allergic to any of the ingredients.

Side effects

In singular cases, Harvard TEMP Cem EasyDose® may cause a sensitizing reaction in patients with a hypersensitivity to any of the ingredients. In these cases, the material should not be used.

Application
1. Preparing the EasyDose®-syringe

Remove the cap of the EasyDose®-syringe and put it aside. **(Do not throw it away!)**

When using the syringe the first time, press the plunger gently until the material flows evenly from both orifices and discard this material.

Extrude the desired amount of Harvard TEMP Cem EasyDose® with slight and even pressure onto a mixing pad and mix with a suitable spatula for **30 seconds** until the material gets to a homogeneous consistency.

After use close the EasyDose®-syringe with the cap.

2. Application

Apply a thin layer Menge Harvard TEMP Cem EasyDose® to the inside of the temporary crown or bridge and place the temporary restoration within **1:30 minutes** (working time at 23 °C/74 °F) cautiously but firmly over the prepared tooth structures.

Note:

Do not dry stumps, best moisten them with some water. A humid milieu (humid stumps) additionally facilitates to remove the temporary crown or bridge later on.

Harvard TEMP Cem EasyDose® sets in the mouth within **about 2-3 minutes**. After Harvard TEMP Cem EasyDose® has set remove any excess.

3. Precautions

- Avoid contact of Harvard TEMP Cem EasyDose® with skin and eyes.

- In case of contact with skin and eyes rinse immediately with plenty of water and consult a physician if necessary.
- If swallowed seek medical advice.

Storage information

Store at 8 – 25 °C (47 - 77 °F).
Do not use after expiry date.

Warranty

Harvard Dental International GmbH warrants this product will be free from defects in material and manufacture. Harvard Dental International GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and Harvard Dental International GmbHs sole obligation shall be repair or replacement of the Harvard Dental International GmbH product.

Limitation of liability

Except where prohibited by law, Harvard Dental International GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

For dental use only!
Store product out of reach of children!

Harvard TEMP Cem EasyDose® meets the requirements of ISO 3107 and the ADA/ANSI Specification No. 30. Keep the instructions for use for the duration of the application.

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